

## Maintenance & Care Tips

### ■ Upholstery Care

To prolong the life of upholstery we recommend regularly wiping cushions with a damp cloth containing a mild soap and water solution.

### ■ Locating the Serial Number

The serial number is located at the foot end of the table as shown in the illustrations below. You will need this number when making inquiries to ELITE customer support.



### ■ Moving Your Table

We recommend having two people to move the table. One person placed at each end of the table. The following will illustrate the proper lift points. The tables are heavy, only attempt moving the table if it is safe to do so.

The person lifting from the front of the table should push the headrest **Tilt Release Knob** in to lock the upward motion of the headrest. When you apply upward force the headrest should not move.



The person lifting from the back of the table should lift holding onto the handles as shown below.



<b>ELITE</b>	<b>MADE IN CANADA</b>
<b>MANUFACTURING</b>	FOR SALES OR SERVICE
Division of 1180396 Ontario Inc.	TEL: (905) 768-0550 FAX: (905) 758-0822
692 Nanticoke Creek Parkway, Jarvis, Ontario, Canada, N0A 1J0	
SER. # <input type="text"/>	PO. # <input type="text"/>
MODEL <input type="text"/>	
<b>CHIROPRACTIC TABLE</b>	

# TABLE OVERVIEW

## Drop Maintenance & Troubleshooting

It is very important to clean and lubricate the drop pieces on the table every 3 months. If you are using the drop pieces more frequently we recommend maintaining the drops every 2 months. The following steps can be done preventatively or in the event the drop is sticking.

**Step 1 - Using a clean dry rag, remove any existing lubricant from the drop piece.**

**Step 2 - Using a small paint brush and petroleum jelly or Vaseline, apply a thin layer of lubricant to the drop piece in the locations shown below.**

**IMPORTANT - Do not use white grease.**

This will prevent sticking and prolong the life of your drop.

It is also recommended to loosen off the tension on the drop pieces at the end of each working day to prolong the life of your drop springs.

